

IMPORTANT – PLEASE READ!

PROLONGED VIBRATION

ADAMM-RSM™ vibrates for a few seconds as part of the Powering OFF procedure. If ADAMM-RSM™ continues to vibrate for 6 or more seconds, please follow the procedure indicated below.

To stop the vibration, *Press and Hold* Power Button **2** for 15 seconds. The vibration will stop during this time. If the vibration does not stop, release the Power Button **2** and repeat this procedure, ensuring that you are not releasing the button during the count to 15 seconds.

You may restart the device and use at any time afterwards.

